



By Appointment to
Her Majesty The Queen
Supply and Maintenance
of Horse Weighing Equipment

Horse Weigh®

Tokyo Golden Rules

1. Always assemble/breakdown on a flat surface & with the rubber side downmost.

2. Assembly

- i.** Place end "1" to middle section "1" – keep safety pin backwards.
- ii.** Place end "2" to middle section "2" – keep safety pin backwards.
(Please note – the "1's" & "2's" are clearly marked on the underneath of the platform.)
- iii.** Firstly attach the 2 outer handles, followed by the middle handle, and then insert the 3 safety pins.

3. Breakdown

- i.** Take the 3 safety pins out and place backwards.
 - ii.** Undo the centre handle and lift 'T' section backwards.
 - iii.** Lift the 2 outer handles and push the 2 'T' sections against the black pads and continue to push the 2 handles away from you until the end section separates itself from the middle section.
 - iv.** Lift the 2 'T' sections backwards.
 - v.** Lift end section away and lay the 3 'T' sections flat and re-insert the 3 safety pins. Repeat these 5 points for the other end section.
- 4.** If the gaps/spaces by any of the handles increase at any time, then as viewed from the top, loosen the nut anticlockwise with the supplied spanner, turn the 'T' section clockwise – half a turn, and then tighten the nut clockwise.

5. If you are weighing on any soft surface or ground, the 4 supplied aluminum plates **MUST BE USED** prior to weighing commencing. Place 1 plate under each **BLACK RUBBER FOOT**. For horse safety, please make sure all 4 aluminum plates are placed level to the platform outer edges, and making sure that they are not protruding outside the platform edges.